

Multi-Panel Saliva Screening kit



Oral Fluid Screening Kit Drug testing made simple

- Observed collection, difficult to fake
- Simple 2 piece kit
- 18-24 month shelf life
- Easy to use

Multi Panel Saliva Screening Kit

This is a rapid screening test for the simultaneous detection of drugs in human saliva.




Benefits


- Simple 2 piece package
- 18-24 month shelf life
- Observed collection
- Lowest industry price

Two options available in stock:
 THC/COC/OPI/AMP/MAMP/PCP
 THC/COC/OPI/MAMP/BZO/OXY

COLLECTION PROCEDURES

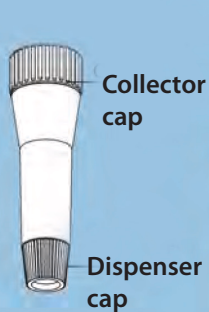

1. Bring the pouch to room temperature, remove test and use as soon as possible. Unscrew the collector cap and remove the sponge.

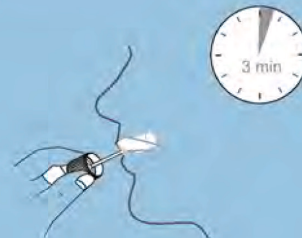


2. Insert the sponge in the mouth and swap for 3 minutes. The sponge will be free of hard spots when enough oral fluid is collected.





3. Place the testing device on a level surface and insert the sponge back into the collection tube.



4. Unscrew the dispenser cap.

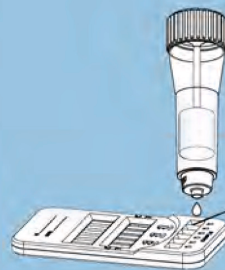

5. Squeeze the tube toward the specimen test well. Place 1 drop of fluid in each well, then read results in 5 minutes.

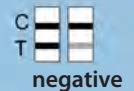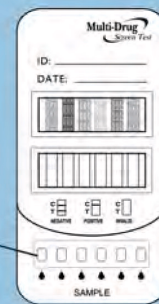

Test is designed as single use only